


koolRAP-knee™



The koolRAP-knee™ is used for post-operative and post-injury care. The koolRAP-knee™ is made from the finest quality of stretch absorbent material that features velcro closures to adjust desired fit. It is specially designed to hold the koolPAK™ (6" x 9" or 6" x 12") in place for cold therapy/pain relief without harsh medical tapes.

- Tapeless support to hold koolPAK™ in place
- Eliminates tape blisters
- Gentle compression
- Easy access pocket for cold therapy
- Various sizes available
- Latex-free

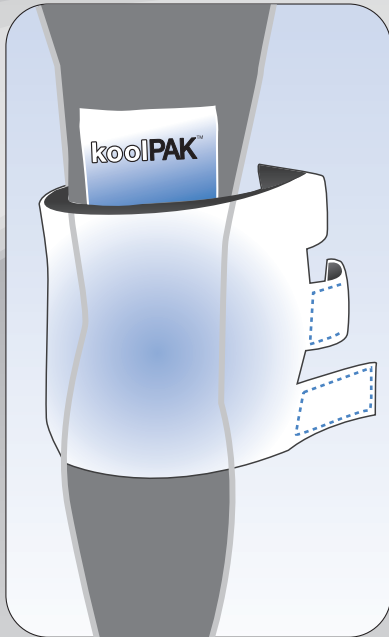

- koolPAK™ cold therapy placement



- Pocket access area between layers of fabric



- Velcro closures to adjust fit

INSTRUCTIONS

Insert the koolPAK™ into the special designed pocket (FIGURE 1) and wrap the koolRAP™ around the knee. Attach the hook and loop closure. *Machine wash/gentle cycle with cold water. Line dry and reuse.*

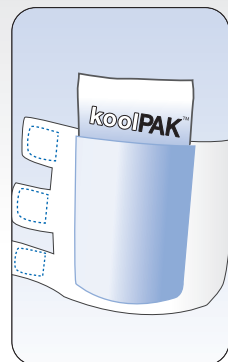


FIGURE 1

PRECAUTIONS

Improper sizing may result in skin irritation. Use sizing chart to select appropriate size. This product has not been sterilized. A sterile dressing should be applied (if treating a wound) between the wrap and the actual wound. This product **SHOULD NOT** be utilized if the patient is allergic to cotton, lycra, dacron, nylon thread. Do not use this wrap other than what is specified.

*For more information, please contact us
and we will be happy to assist you.*



www.sunmedica.com

P 530.229.1600 | **F** 530.229.9457 | **TOLL FREE** 1.800.995.8715
1661 Zachi Way, Redding, CA 96003