



koolRAP-uni™

*ankle, shoulder, back, neck, wrist,
hand, quadricep, calf*

The koolRAP-uni™ (universal) is a convenient and comfortable way to hold cold therapy packs in place. It's design also allows you to be mobile while "icing down". This koolRAP-uni™ can be used for post-op rehabilitation and is also great for sports injuries or sore/achy muscles that need pain relief. The koolRAP-uni™ is available in two sizes to accommodate a 6" x 9" or 6" x 12" koolPAK™. The koolRAP-uni™ can be applied to the ankles, shoulders, back, neck, wrist, hand, quaudricep, calf, etc.)

- Allows mobility while applying cold therapy
- Available in two sizes 6" x 9" & 6" x 12"
- Latex-free (except straps)
- 4 straps for right fit 8", 10", 15", 20"



• *koolPAK™ cold therapy placement*



• *Velcro closures to adjust fit*

INSTRUCTIONS

1. Insert the koolPAK™ cold therapy pack into the koolRAP-uni™ pocket.
2. Place the cotton side of the wrap next to the skin.
3. Using one or more of the straps secure the wrap in the desired area by attaching the hook to the textured fabric.

PRECAUTIONS

This product SHOULD NOT be utilized if the patient is allergic to cotton, polyester, or nylon thread.

*For more information, please contact us
and we will be happy to assist you.*



www.sunmedica.com

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715
1661 Zachi Way, Redding, CA 96003