


# shoulderRAP™



The shoulderRAP™ is used for post-operative, post-injury patient care. The shoulderRAP™ is made from the finest quality of stretch absorbent material. It provides soft tissue support and secures sterile dressings in place without the use of harsh medical tapes. It's unique design features a special pocket to hold the koolPAK™ in place for cold therapy/pain relief.

- Tapeless support of post operative dressing
- Eliminates tape blisters
- Greatly improves patient comfort
- Can reduce post-operative complications
- Decreases wound care time
- Latex-free
- Easy access pocket for cold therapy

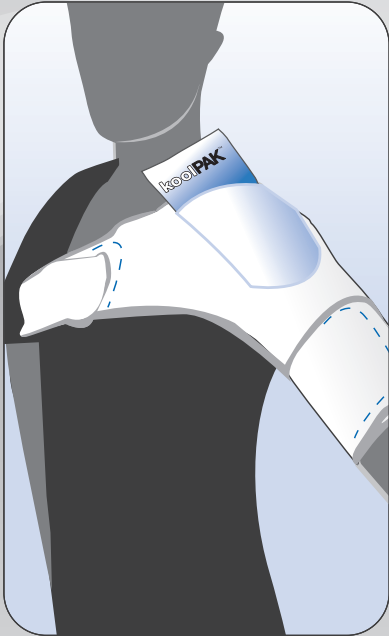

• *koolPAK™ cold therapy placement*



• *Pocket access area between layers of fabric*



• *Velcro closures to adjust fit*

## INSTRUCTIONS

1. The shoulderRAP™ is placed directly over the lateral shoulder/arm areas. Apply so you can read “right” when the RAP is applied to the right shoulder and “left” when applied to the left shoulder.
2. Adjust the shoulderRAP™ so the axillary area has ample space to prevent compression of the axillary structures.
3. Slide the chest portion around the back and under the opposite arm so that the chest hook and loop attachments are opposed. Remove the felt from the hook and attach the loop (see FIGURE 1).

4. Apply the posterior arm portion across the back of the arm around the medial arm to oppose the hook and loop attachment. Pull the felt from the hook and secure. Use the natural contours of the patient to guide the position of the hook/loop attachments so the support is uniform.
5. Smooth and adjust the shoulderRAP™ so there are no wrinkles and edges do not bunch. Adjust the wrap often to ensure comfort.

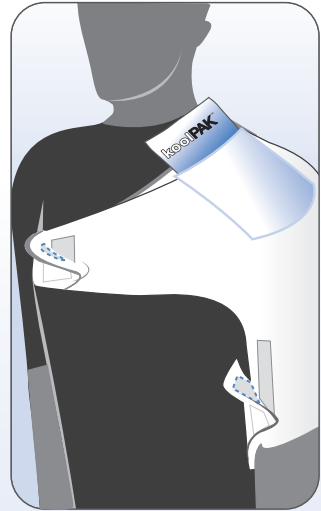


FIGURE 1

## PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a **MEDICAL PROFESSIONAL ONLY**. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are **NOT STERILE. DO NOT APPLY DIRECTLY OVER WOUND**. The sizing chart is approximate, the optimal size may need to be determined at the time of application. **DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.**

*For more information, please contact us  
and we will be happy to assist you.*



[www.sunmedica.com](http://www.sunmedica.com)

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715

1661 Zachi Way, Redding, CA 96003

# shoulderRAP™

ORDER #	SIZE	CHEST DIAMETER
053-11-P	Petite	32"-36"
053-11-S	Small	36"-40"
053-11-M	Medium	40"-44"
053-11-L	Large	44"-50"
053-11-XL	X-Large	50"-56"

*Measure chest at Axila, do not include breast.*