



legGRIP™ FOOT SUPPORT & KNEE SUPPORT Foam Pad

Supplies Needed: SOAP OR WAX

TO REMOVE/REPLACE:

- Remove old pad from support.
- Clean and lubricate surface with soap or wax (Figure 1).
- Place the thick side of the pad on the inside (opposite of the **Support Rod** which is the side toward the patient) and slide on (Figure 2). This applies to both types of pads (Knee Support featured).

*For more information or to re-order,
you can visit www.sunmedica.com or
give us a call and we will be happy
to assist you!*

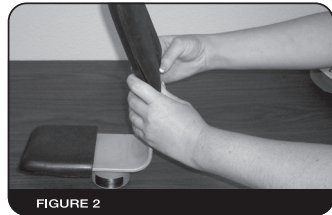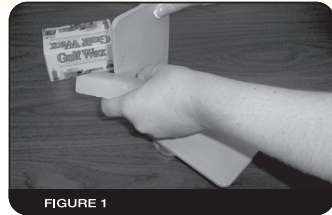

NOTE: *Thick side of pad
toward patient*



www.sunmedica.com

P 530.229.1600 | 1.800.995.8715 | F 530.229.9457

1661 Zachy Way, Redding, CA 96003