
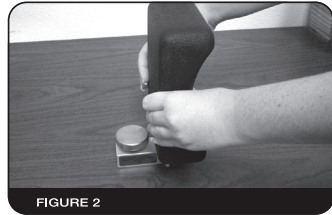


# Upright Thigh Support Foam Pad

**Supplies Needed:  
SOAP OR WAX**

## **TO REMOVE/REPLACE:**

- Remove old pad from support.
- Clean and lubricate surface with soap or wax (Figure 1).
- Place the thick side of pad on side opposite of thumb screw and slide on (Figure 2).

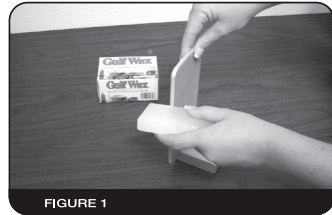

*For more information or to re-order,  
you can visit [www.sunmedica.com](http://www.sunmedica.com) or  
give us a call and we will be happy to  
assist you!*



**[www.sunmedica.com](http://www.sunmedica.com)**

P 530.229.1600 | 1.800.995.8715 | F 530.229.9457  
1661 Zachi Way, Redding, CA 96003