



*Dimensions: 9 1/2" x 15"*

# backRAP™ Extension

## Back Extension for the Difficult-to-Fit Body Shapes

The backRAP™ Extension attaches to the backRAP™, allowing a proper fit on extra large patients.

- Tapeless support of post operative dressings
- Eliminates tape blisters
- Greatly improves patient comfort
- Can reduce complications
- Decreases wound care time
- Latex-free

## INSTRUCTIONS

1. Extensions are to be used for the larger shaped bodies. They should be utilized for a better fit if the initial backRAP™ does not fit correctly.
2. Attach the extension at the waist by using the hook and loop attachment (Figure 1). For application of the backRAP™, please see backRAP™ instructions within product box.
3. Adjust the tension so that it is not constrictive or uncomfortable (Figure 2).



FIGURE 1



FIGURE 2

## PRECAUTIONS

*Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a MEDICAL PROFESSIONAL ONLY. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are NOT STERILE. DO NOT APPLY DIRECTLY OVER WOUND. The sizing chart is approximate, the optimal size may need to be determined at the time of application. DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.*

## WASHING INSTRUCTIONS

*Wash in cold water and hang dry.*

*For more information, please contact us  
and we will be happy to assist you.*



**www.sunmedica.com**

**P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715**  
1661 Zachi Way, Redding, CA 96003