


wristRAP™

WITH OPENING & PULL-ON

ORDER# 014-11-size | ORDER# 010-11-size

The wristRAP™ is used for post-injury or post-surgery care. The wristRAP™ is made from the finest quality of stretch absorbent material and provides gentle soft tissue support. It secures sterile dressings in place without the use of harsh medical tapes.

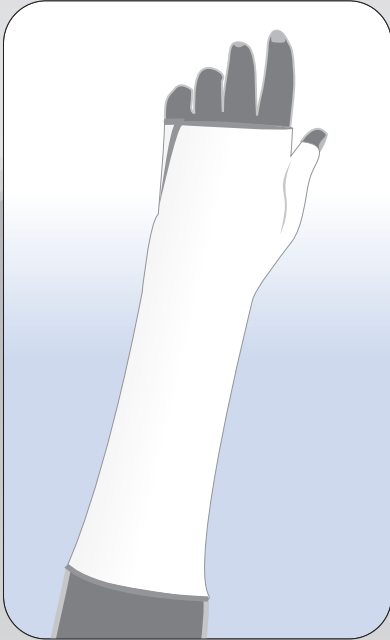

- Tapeless support of post-operative dressing
- Eliminates tape blisters
- Greatly improves patient comfort
- Can reduce post-operative complications
- Decreases wound care time
- Latex-free

INSTRUCTIONS

wristRAP™ WITH OPENING

1. Place a sterile dressing over the surgical wound.
2. Place the wristRAP™ on the wrist securing the sterile dressing.
3. Close the opening (FIGURE 1).

See back of instructions for wristRAP™ Pull-On instructions.

⋮ *Velcro closures to adjust fit*

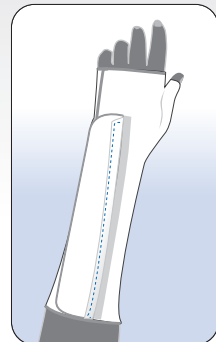


FIGURE 1

wristRAP™ PULL-OVER

1. Place a sterile dressing over the surgical wound.
2. Place the wristRAP™ on the wrist securing the sterile dressing.
3. Slide the wristRAP™ over the dressing (FIGURE 2).

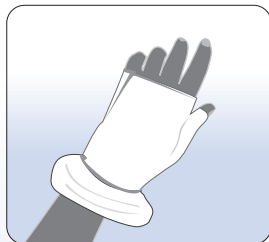


FIGURE 2

PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a **MEDICAL PROFESSIONAL ONLY**. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are **NOT STERILE**. **DO NOT APPLY DIRECTLY OVER WOUND**. The sizing chart is approximate, the optimal size may need to be determined at the time of application. **DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.**

*For more information, please contact us
and we will be happy to assist you.*



www.sunmedica.com

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715
1661 Zachi Way, Redding, CA 96003

wristRAP™

ORDER #	SIZE	WRIST MEASUREMENT
014-11-S	Small	5 1/2"-6 1/2"
014-11-M	Medium	7"-8"
014-11-L	Large	8 1/2"-9 1/2"

With Opening

wristRAP™

ORDER #	SIZE	WRIST MEASUREMENT
010-11-S	Small	5 1/2"-6 1/2"
010-11-M	Medium	7"-8"
010-11-L	Large	8 1/2"-9 1/2"

Pull-on