

8" Upright Support Foam Pad

Supplies Needed:
SOAP OR WAX

TO REMOVE:

1. **Pull Foam Pad** off of the **Upright Support**.
2. Clean **Upright Support** with mild soap solution. (NO ADHESIVE IS USED ON **UPRIGHT SUPPORT**)
3. LET DRY THOROUGHLY.

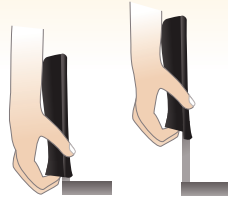


FIGURE 1

NOTE: *Thick side of pad toward patient*

TO REPLACE:

1. Rub SOAP OR WAX on the edges of the **Upright Support**.
2. Pull **NEW FOAM PAD** onto the **Upright Support**, like placing a stocking on a foot, ie; grasp open end and pull over support. (SEE FIGURE 1) **THICK PART OF THE PAD MUST BE ON THE SIDE CONTACTING THE PATIENT.**
3. Clean any excess wax or soap from exposed area of the support.
4. Apply pressure on both sides of the pad to ensure good conduct between the surface of the support and the pad.

*For more information or to re-order,
you can visit www.sunmedica.com or give us a call
and we will be happy to assist you!*



www.sunmedica.com

P 530.229.1600 | 1.800.995.8715 | F 530.229.9457
1661 Zachy Way, Redding, CA 96003