
### orthoRAPs®

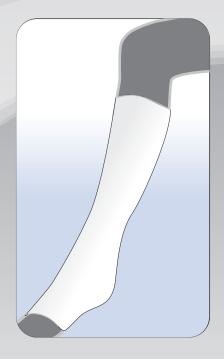

# amkleRAP"

### WITH OPENING & PULL-ON

ORDER# 015-11-size | ORDER# 012-11-size

The ankleRAP™ is used for post-injury or post-surgery care. The ankleRAP™ is made from the finest quality of stretch absorbent material and provides gentle soft tissue support. It secures sterile dressings in place without the use of harsh medical tapes.

- Tapeless support of post-operative dressing
- Eliminates tape blisters
- Greatly improves patient comfort
- Can reduce post-operative complications
- · Decreases wound care time
- Latex-free

#### INSTRUCTIONS

#### ankleRAP™WITH OPENING

- 1. Place a sterile dressing over the surgical wound.
- 2. Place the ankleRAP™ on the ankle securing the sterile dressing.
- 3. Close the opening (FIGURE 1).

See back for ankleRAP™ Pull-On instructions.

#### Velcro closures to adjust fit



FIGURE 1

#### ankleRAP™ PULL-OVER

- 1. Place a sterile dressing over the surgical wound.
- Slide the ankleRAP<sup>™</sup> over the foot and pull up over the ankle securing the sterile dressing (FIGURE 2).



FIGURE 2

#### **PRECAUTIONS**

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a MEDICAL PROFESSIONAL ONLY. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are NOT STERILE. DO NOT APPLY DIRECTLY OVER WOUND. The sizing chart is approximate, the optimal size may need to be determined at the time of application. DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.

For more information, please contact us and we will be happy to assist you.



www.sunmedica.com P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715 1661 Zachi Way, Redding, CA 96003

## ankleRAP"

ORDER#	SIZE	WOMEN SHOE	MEN SHOE	With
015-11-S	Small	4-5	3-5	==
015-11-M	Medium	5-8	6-9	Oper
015-11-L	Large	9 & UP	10-12	ening

## ankleRAP\*

ORDER#	SIZE	WOMEN SHOE	MEN SHOE
012-11-S	Small	4-5	3-5
012-11-M	Medium	5-8	6-9
012-11-L	Large	9 & UP	10-12

Pull-on