

orthoRAPs®

ORDER # 060-11-Size
Petite/Small, Medium/Large



backRAP™

The backRAP™ is used for post-injury or post-surgery care. The backRAP™ is made from the finest quality of stretch absorbent material and provides gentle soft tissue support. It secures sterile dressings in place without the use of harsh medical tapes. It's unique design features a special pocket to hold the koolPAK™ in place for cold therapy/pain relief.

- Tapeless support of post operative dressings
- Eliminates tape blisters
- Easy access pocket for cold therapy
- Can reduce complications
- Decreases wound care time
- Latex-free
- Greatly improves patient comfort

INSTRUCTIONS

1. Place a sterile dressing over the surgical wound.
2. The backRAP™ is placed directly over the lower back and abdominal area. The wide portion with the pocket is placed over a sterile dressing on the low back or abdomen. The attachment is over the abdominal area in the front of the patient. See FIGURE 1.
3. Adjust the tension on the anterior attachment for support, but not so tight that it is constrictive or uncomfortable. See Figure 2.



FIGURE 1



FIGURE 2

4. Place a koolPAK™ in between the layers of fabric. See Figure 3.
5. Smooth and adjust the backRAP™ so that there are no wrinkles and that the edges do not bunch. Adjust the wrap often to ensure comfort. See Figure 4.



FIGURE 3

Order #	Size	Weight
060-11-P/S	Petite/Small	90-140 LBS
060-11-M/L	Medium/Large	140-220 LBS
*060-11-EXT Extension	9 1/2" x 15" (1 per order)	Attaches to the Large & XL Sizes



FIGURE 4

**The backRAP™ Extension connects to the Medium/Large size to accommodate obese patients. Find out more at www.sunmedica.com.*

PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a MEDICAL PROFESSIONAL ONLY. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are NOT STERILE. DO NOT APPLY DIRECTLY OVER WOUND. The sizing chart is approximate, the optimal size may need to be determined at the time of application. DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.

WASHING INSTRUCTIONS

Wash in cold water and hang dry.

*For more information, please contact us
and we will be happy to assist you.*



www.sunmedica.com

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715

1661 Zachy Way, Redding, CA 96003

PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a MEDICAL PROFESSIONAL ONLY. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are NOT STERILE. DO NOT APPLY DIRECTLY OVER WOUND. The sizing chart is approximate, the optimal size may need to be determined at the time of application. DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.

WASHING INSTRUCTIONS

Wash in cold water and hang dry.

*For more information, please contact us
and we will be happy to assist you.*



www.sunmedica.com

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715
1661 Zachy Way, Redding, CA 96003



Dimensions: 9 1/2" x 15"

backRAP™ Extension

Back Extension for the Difficult-to-Fit Body Shapes

The backRAP™ Extension attaches to the backRAP™, allowing a proper fit on extra large patients.

- Tapeless support of post operative dressings
- Eliminates tape blisters
- Greatly improves patient comfort
- Can reduce complications
- Decreases wound care time
- Latex-free

INSTRUCTIONS

1. Extensions are to be used for the larger shaped bodies. They should be utilized for a better fit if the initial backRAP™ does not fit correctly.
2. Attach the extension at the waist by using the hook and loop attachment (Figure 1). For application of the backRAP™, please see backRAP™ instructions within product box.
3. Adjust the tension so that it is not constrictive or uncomfortable (Figure 2).



FIGURE 1



FIGURE 2