

orthoRAPs®
ORDER # 003-11-Ext.



Dimensions: 7 3/4" x 12 1/2"
(Comes with 2 per order)

hipRAP™ Extension

Waist & Thigh Extensions for the Difficult-to-Fit Body Shapes

The hipRAP™ Extension will allow a proper fit of the hipRAP™ on extra large patients.

- Tapeless support of post operative dressings
- Eliminates tape blisters
- Greatly improves patient comfort
- Accommodates obese patients
- Can reduce complications
- Decreases wound care time
- Latex-free

INSTRUCTIONS

1. Extensions should be used to allow appropriate application of the hipRAP™ without causing tightness or irritation to the thigh. They are designed to be attached to the waist and thigh portions of the Large & XL hipRAP™. See FIGURE 4.
2. Attach the extension at the waist and/or thigh by using the hook and loop attachments as shown above. For application of the hipRAP™, please see the hipRAP™ instructions within the product box. See FIGURE 1.



FIGURE 1

PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a MEDICAL PROFESSIONAL ONLY. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are NOT STERILE. DO NOT APPLY DIRECTLY OVER WOUND. The sizing chart is approximate, the optimal size may need to be determined at the time of application. DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.

WASHING INSTRUCTIONS

Wash in cold water and hang dry.

*To view sizing charts, order numbers and the latest pricing,
visit www.sunmedica.com.*



www.sunmedica.com

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715
1661 Zachi Way, Redding, CA 96003