

orthoRAPs®

ORDER # 013-11-Size
Petite, Small, Medium, Large & XL



kneeRAP™

The kneeRAP™ is applied immediately after surgery and is used for post-operative wound care. The kneeRAP™ is made from the finest quality of stretch absorbent material. It provides soft tissue support and secures sterile dressings in place without the use of harsh medical tapes. It's unique design features a special pocket to hold the koolPAK™ in place for cold therapy/pain relief.

- Tapeless support of post operative dressings
- Eliminates tape blisters
- Greatly improves patient comfort
- Velcro attachments allow for easy wound checks
- Easy access koolPAK™ pocket
- Can reduce complications
- Decreases wound care time
- Latex-free
- Reversible to “right” or “left”

INSTRUCTIONS

1. The kneeRAP™ is rolled into a “donut” shape (FIGURE 1) starting from the large (proximal) end down to the bottom of the cold therapy pocket.
2. Pull the kneeRAP™ over the foot, ankle and lower leg until the bottom of the cold therapy pocket is just below the knee. The seam should be posterior (FIGURE 2).
3. Roll the kneeRAP™ over the knee and thigh. A dressing should be placed between the wrap and any incision or wounds.
4. Smooth and adjust the wrap so that there are no wrinkles or bunched edges. Adjust the wrap often to ensure comfort.
5. Place a koolPAK™ in between the layers of fabric for cold therapy/pain relief. See FIGURE 3.



FIGURE 1



FIGURE 2



FIGURE 3

kneeRAP™

Order #	Size	Knee Circumference
013-11-P	Petite	11"-13"
013-11-S	Small	13"-16"
013-11-M	Medium	16"-19"
013-11-L	Large	19"-21"
013-11-XL	XLarge	21"-23"

PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a MEDICAL PROFESSIONAL ONLY. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are NOT STERILE. DO NOT APPLY DIRECTLY OVER WOUND. The sizing chart is approximate, the optimal size may need to be determined at the time of application. DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.

WASHING INSTRUCTIONS

Wash in cold water and hang dry.



www.sunmedica.com
P 530.229.1600 | TOLL FREE 1.800.995.8715
1661 Zachary Way, Redding, CA 96003