


Giving women the comfort they deserve...



masRAP™

The masRAP™ is used for post-breast surgery care (breast removal, reconstruction, reduction or enlargement). It is made from the finest quality of stretch-absorbent material, provides gentle soft tissue support and secures sterile dressings in place without the use of harsh medical tapes. The masRAP™ also features special designed pockets on the left and right breast to hold the koolPAK™ in place for cold therapy/pain relief.

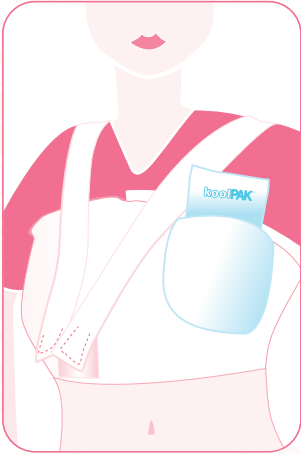

• **koolPAK™** cold therapy placement



• **Pocket access area** between layers of fabric



• **Velcro®** closures to adjust fit

- Tapeless support of post-operative dressings
- Helps to eliminate tape blisters
- Greatly improves patient comfort
- Easy access pockets for cold therapy
- Can reduce complications
- Decreases wound care time
- Latex-free
- Discreet under clothing

orthoRAP®

OR017 R7

masRAP™

- A koolPAK™ pocket areas
- B Velcro® attachment area

Instructions

1. Place the masRAP™ around the chest and attach the Velcro® closure (FIGURE 1).
2. Pull the straps over the shoulders and attach to the hook that is under the felt tab. If straps are not wanted, cut them off and press the felt over the hook to protect against snagging of clothing and scratching of skin (FIGURE 2).
3. The special designed pockets on the right and left breast are easily accessible to slide cold therapy packs in between the layers of fabric (FIGURE 2).
4. Adjust as needed to ensure even pressure over the surgical site.

FIGURE 1


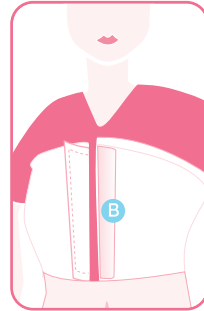


FIGURE 2

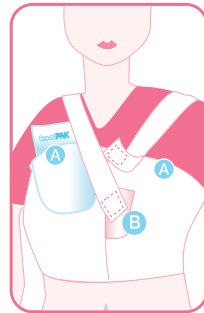

PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a MEDICAL PROFESSIONAL ONLY. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are NOT STERILE. DO NOT APPLY DIRECTLY OVER WOUND. The sizing chart is approximate, the optimal size may need to be determined at the time of application. DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.

View our complete
catalog & online store at
www.sunmedica.com

*For more information you can give us a call
and we will be happy to assist you.*



www.sunmedica.com

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715

1661 Zachi Way, Redding, CA 96003

OR017 R7

masRAP™

ORDER #	SIZE	CHEST DIAMETER
016-11-P	Petite	24"-30"
016-11-S	Small	28"-32"
016-11-M	Medium	30"-36"
016-11-L	Large	34"-42"
016-11-XL	X-Large	38"-48"