

orthoRAPs®

ORDER # 017-11-Size
Small, Medium, Large, XL



tummyRAP™

The tummyRAP™ is used for post-operative wound care. It is made from the finest quality of stretch absorbent material and provides gentle soft tissue support. It secures sterile dressings in place without the use of harsh medical tapes and features a special pocket to hold the koolPAK™ in place for cold therapy/pain relief.

- Tapeless support of post operative dressings
- Eliminates tape blisters
- Easy access pocket for cold therapy
- Can reduce complications
- Decreases wound care time
- Latex-free
- Greatly improves patient comfort

INSTRUCTIONS

1. Place a sterile dressing over the surgical wound.
2. The tummyRAP™ is placed directly over the lower stomach and abdominal area. The wide portion with the pocket is placed over a sterile dressing on the low stomach or abdomen. The attachment is over the hip area on the side of the patient. See FIGURE 1.
3. Adjust the tension on the anterior attachment for support, but not so tight that it is constrictive or uncomfortable. See Figure 2.



FIGURE 1



FIGURE 2

- Place a koolPAK™ in between the layers of fabric. See Figure 3.
- Smooth and adjust the tummyRAP™ so that there are no wrinkles and that the edges do not bunch. Adjust the wrap often to ensure comfort. See Figure 4.



FIGURE 3



FIGURE 4

Order #	Size	Around Waist
017-11-S	Small	22"-34"
017-11-M	Medium	27"-41"
017-11-L	Large	33"-53"
017-11-XL	XLarge	40"-60"
017-11-EXT Extension	9 1/2" x 15" (1 per order)	Attaches to Large & XL Sizes

PRECAUTIONS

Our orthoRAPs® are not designed for any other use except as indicated. These wraps should be applied properly by a MEDICAL PROFESSIONAL ONLY. The patient should be frequently monitored for signs of discoloration or circulatory problems. These products are NOT STERILE. DO NOT APPLY DIRECTLY OVER WOUND. The sizing chart is approximate, the optimal size may need to be determined at the time of application. DO NOT USE ON PATIENTS WITH ALLERGIES TO COTTON OR LYCRA.

WASHING INSTRUCTIONS

Wash in cold water and hang dry.

To view sizing charts, order numbers and the latest pricing, visit www.sunmedica.com.



www.sunmedica.com

P 530.229.1600 | F 530.229.9457 | TOLL FREE 1.800.995.8715

1661 Zachi Way, Redding, CA 96003