

**hipGRIP®**

ORDER # D-030-006  
Includes 2 Foam Pads

# Anterior Pelvic Support Foam Pad Set

*Supplies Needed:*  
SPRAY ADHESIVE (77™ BY 3M™ IS RECOMMENDED)  
& ADHESIVE REMOVER

## TO REMOVE:

1. **Pull Foam Pads** off of the Anterior Pelvic Support.
2. Clean with adhesive remover.
3. LET DRY THOROUGHLY.



FIGURE 1

**NOTE:** Thick side of pad toward patient

## TO REPLACE:

1. Spray adhesive on the inside of the replacement pad.
2. Push NEW **Foam Pads** onto the Anterior Pelvic Support. (SEE FIGURE 1) THICK PART OF THE PAD MUST BE ON THE SIDE CONTACTING THE PATIENT.
3. Apply pressure on BOTH SIDES of the pad to ensure good contact between the surface of the support and the pads.
4. Clean any excess adhesive from the exposed surface of the support.

*For more information or to re-order,  
you can visit [www.sunmedica.com](http://www.sunmedica.com) or give us a call  
and we will be happy to assist you!*



[www.sunmedica.com](http://www.sunmedica.com)

P 530.229.1600 | 1.800.995.8715 | F 530.229.9457

1661 Zachi Way, Redding, CA 96003