


Upright Thigh Support Foam Pad

**Supplies Needed:
SOAP OR WAX**

TO REMOVE/REPLACE:

- Remove old pad from support.
- Clean and lubricate surface with soap or wax (Figure 1).
- Place the thick side of pad on side opposite of thumb screw and slide on (Figure 2).

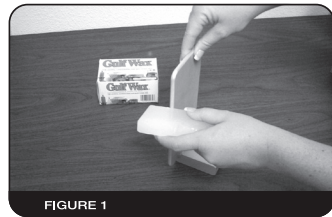

*For more information or to re-order,
you can visit www.sunmedica.com or
give us a call and we will be happy to
assist you!*



www.sunmedica.com

P 530.229.1600 | 1.800.995.8715 | F 530.229.9457
1661 Zachi Way, Redding, CA 96003